



Lucid[™] AUDIO[™]

AMPED[™] Bluetooth[®] Headphones
(HLT-BT-H-PH Series)

Easy Start Guide

Turning the Device ON/OFF

Roll the Power knob upwards until you hear the click to turn the unit on OR downwards till you hear the click to turn the unit off.

Adjusting Outside Ambient Sound Level

Roll the Volume knob upwards to increase the outside ambient sound level or downwards to decrease the outside ambient sound level.



Power / Volume Knob

Pairing First Bluetooth® Device

Press and hold the Bluetooth® Sync button for 1 second to enter pairing mode after device is turned on (this will time out after 60-seconds).

Open Bluetooth® wireless setting on your phone or mp3 player.

Make sure Bluetooth® is on and select "Lucid BT" to connect.

Auto re-connects to previous last connected device during pairing if the BT device has been paired previously.

Once paired, blue LED will be flashing every 10 seconds.



Bluetooth® LED

Bluetooth® Sync button

Adjusting Bluetooth® Volume

Roll the Bluetooth® Volume knob upwards to increase the Bluetooth® device sound level or downwards to decrease the Bluetooth® device sound level.

Music Audio Pause/Play

Short press the Bluetooth® Sync button to pause/play your music.



Bluetooth® Volume Knob

Bluetooth® Sync button

Phone Call Functions

Answering Call: Short press BT button

Rejecting Call: Long press BT button

Cancelling or Ending Call: Short press BT button

Last Number Dialed: Double press BT button



Bluetooth® Sync button