



# PREMIER BTE USER MANUAL



## Tips



1. Place tip on tube
2. Push down over tube ribbing
3. Make sure tip is secure
4. To remove, gently pull off tube

## Tube Selection

Choose left or right tube

**Attachment:** Place tube over brass tip of unit and press down gently.

**Removal:** Grasp base of tube and gently pull.



TIP: Keep unit dry - remove when bathing

Tip: Clean tube by detaching gently and using cleaning tool to push out debris

## BATTERY INFORMATION

**INSERTION:** Remove adhesive sticker on size 312 battery. Open battery door and place battery with + side upward. Closing the door will activate the unit in Program 1.

**REMOVAL:** Open the battery door and lift the size 312 battery out.

## POWER

**POWER ON:** close battery door with battery inserted.

**POWER OFF:** open battery door.



## Ear Placement



1. Grasp unit between your thumb and forefinger



2. Place unit over ear as shown



3. Gently push the ear tip into the ear canal



4. Tuck the concha lock into the bowl of the ear

Low Battery voice prompt indicates time to replace battery

Tip: Remove unit from ear gently to avoid damaging tube

## Program Selection

There are 2 programs to choose from

1 **Default**- "Lucid" voice prompt

2 **Music/Concert** - 2 beeps prompt



Hold rocker switch up/down for 2 seconds to cycle through programs

## About Programs

**Program 1 (Lucid)** will amplify higher frequency sounds and in most situations provide the most comfortable listening

**Program 2 (Music/Concert)** will amplify low frequency sounds

Tip: Test and determine which program sounds best for your situation

## Volume Control

1. Press volume selector up to raise volume

2. Press volume selector down to lower volume

You will hear a quick double tone indicating volume has changed when pressing

Tip: Press and release - do not hold down

## Please contact us with any questions

Email us at -  
[help@lucidaudio.com](mailto:help@lucidaudio.com)

Call us at -  
1 800 775 7124

Chat online and register  
your warranty at -

[www.lucidaudio.com](http://www.lucidaudio.com)

14301 FAA Blvd. Suite 105 Fort Worth, Texas 76155