




PREMIER ITC USER MANUAL

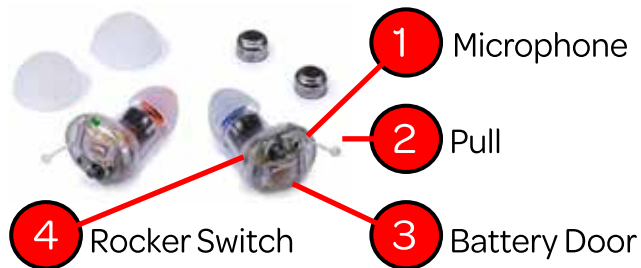

Tips

1. Place tip on unit
2. Push down over colored end
3. Make sure tip is secure
4. To remove, gently pull off unit

Unit Selection

Right Unit: Red tipped unit



Left Unit: Blue tipped unit

Tip: Keep unit dry - remove when bathing

Tip: Clean unit by detaching tip and using cleaning tool to brush debris

BATTERY INFORMATION

INSERTION: Remove adhesive sticker on size 10 battery. Open battery door and place battery with + side upward. Closing the door will activate the unit in Program 1.

REMOVAL: Open the battery door and lift the size 10 battery out.

POWER

POWER ON: close battery door with battery inserted.

POWER OFF: open battery door.



Ear Placement



1. Grasp unit between your thumb and forefinger



2. Place unit over ear as shown



3. Gently push the ear tip into the ear canal



4. When unit is fully inserted, it will be nearly invisible

Low Battery voice prompt indicates time to replace battery

Tip: Remove unit from ear gently to avoid damaging pull

Program Selection

There are 2 programs to choose from

1 **Default**- "Lucid" voice prompt

2 **Music/Concert** - 2 beeps prompt



About Programs

Program 1 (Lucid) will amplify higher frequency sounds and in most situations provide the most comfortable listening

Program 2 (Music/Concert) will amplify low frequency sounds

Hold rocker switch up/down for 2 seconds to cycle through programs

Tip: Test and determine which program sounds best for your situation

Volume Control



1. Press volume selector up to raise volume
2. Press volume selector down to lower volume

You will hear a quick double tone indicating volume has changed when pressing

Please contact us with any questions

Email us at -
help@lucidaudio.com

Call us at -
1 800 775 7124

Chat online and register
your warranty at -

www.lucidaudio.com

Tip: Press and release - do not hold down

14301 FAA Blvd. Suite 105 Fort Worth, Texas 76155