



## ENRICH BTE USER MANUAL



## Tips



1. Place tip on tube
2. Push down over tube ribbing
3. Make sure tip is secure
4. To remove, gently pull off tube

## Tube Selection

Choose left or right tube

**Attachment:** Place tube over brass tip of unit and press down gently.

**Removal:** Grasp base of tube and gently pull.



TIP: Keep unit dry - remove when bathing

Tip: Clean tube by detaching gently and using cleaning tool to push out debris

## BATTERY INFORMATION

**INSERTION:** Remove adhesive sticker on #13 battery. Open battery door and place battery with + side upward. Closing the door will activate the unit in Program 1.

**REMOVAL:** Open the battery door and lift the #13 battery out.

## POWER

**POWER ON:** close battery door with battery inserted.

**POWER OFF:** open battery door.



## Ear Placement



1. Place the Enrich PSAP on the ear. Insert the appropriate size dome tip onto the sound tube at the elbow joint then insert the dome tip into ear.



2. Position the sound tube in an upward direction so that it extends past the ear hook.

Mark the sound tube approx. 1/8" above the tip of the ear hook.



3. Cut off excess tube at mark. Tubing too long will cause the unit to fall out. Tubing too short will pull the tip out of the ear.



4. While holding the sound tube between thumb & forefinger, push the sound tube onto the ear hook. You may need to trim the tube and repeat this process to achieve a comfortable fit.

Low Battery voice prompt indicates time to replace battery

Tip: Remove unit from ear gently to avoid damaging tube

## Program Selection

There are 3 programs to choose from

- 1 General Listening
- 2 Crowds - One tone prompt
- 3 Music/Concert - One tone prompt



Hold rocker switch up/down for 2 seconds to cycle through programs

## About Programs

Program 1 (Lucid) will amplify higher frequency sounds and in most situations provide the most comfortable listening

Program 2 (Crowds) will amplify high frequency sounds in noisy situations

Program 3 (Music/Concert) will amplify low frequency sounds

Tip: Test and determine which program sounds best for your situation

## Volume Control

1. Press volume selector up to raise volume
2. Press volume selector down to lower volume

You will hear a quick double tone indicating volume has changed when pressing



Tip: Press and release - do not hold down

## Please contact us with any questions

Email us at -  
[help@lucidaudio.com](mailto:help@lucidaudio.com)

Call us at -  
1 800 775 7124

Chat online and register  
your warranty at -

[www.lucidaudio.com](http://www.lucidaudio.com)

14301 FAA Blvd. Suite 105 Fort Worth, Texas 76155